



# IMPORTANT TECHNICAL INFORMATION



## PLEASE REVIEW CAREFULLY



To ensure optimal performance and prevent air leakage, please follow these instructions when connecting the cuff to the bayonet:

1. **Align and Insert:** Carefully align the cuff end connection with the bayonet receiver.
2. **Push Firmly:** Push the cuff connection firmly into the bayonet receiver until you hear a distinct "CLICK."
3. **Check Flush Connection:** Visually confirm that the adjoining connectors are flush against each other, as shown in **Example A**.

**Incorrect Connection (Example B):** If the cuff connection is not fully seated, as shown in **Example B**, air leakage may occur.

**Correct Connection (Example A):** A proper connection will be flush and secure, as shown in Example A.

By following these steps, you can ensure a secure connection and prevent air leakage. If you have any questions or concerns, please contact customer support.

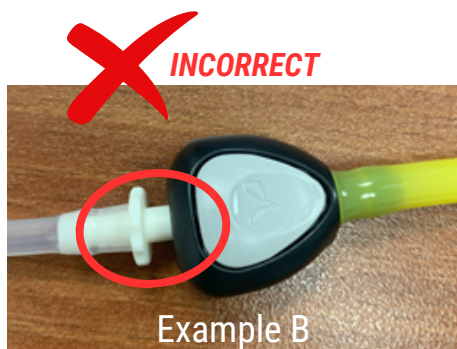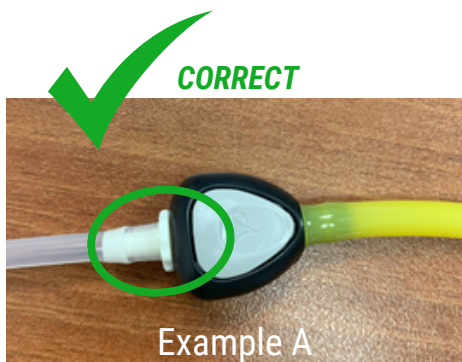

**Customer Support: 800-267-5549**