

# Roll-Stand Assembly Instructions



**1.** Unbox ABI stand and make sure everything is accounted for:

- Cart base
- Computer tray
- Accessories pack:
  - 4 screws
  - 2 Allen wrenches
  - 1 Remote holder



**2.** Turn the black plastic knob ½ turn counter-clockwise and fully extend the shaft.




**3a.** Once fully extended, tighten again.



**3b.** Remove accessories packet from bottom of basket



**3c.** Using the smaller Allen wrench, loosen the Cuff-Link tray so it can move along the shaft.

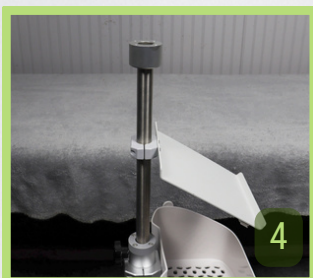


**4.** Lower the Cuff-Link tray about 5 inches from the top of the shaft and then retighten.



**5.** Place computer tray on top of main shaft with tray handle facing forward and align screw holes on shaft head with holds in plate. Hand start each (of the four) screws.



**6.** Continue screwing in each screw with long end on small allen wrench.

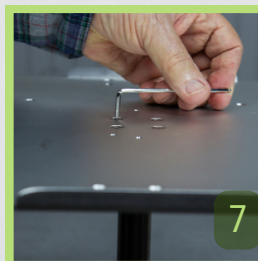

**7.** Finish tightening with short end of allen wrench until snug and then tighten an additional quarter turn.



**8.** Peel protective paper from double sided foam tape on back of remote caddy and place on left side of computer tray. Press down with moderate pressure for ten seconds.



Fully Assembled Stand

Newman  Medical  
**Additional questions?**  
**Call us at:**  
**800-267-5549**

  
[Video Tutorial](#)